



# BD Vacutainer® UltraTouch™ Push Button Blood Collection Set

## Best practice guidelines

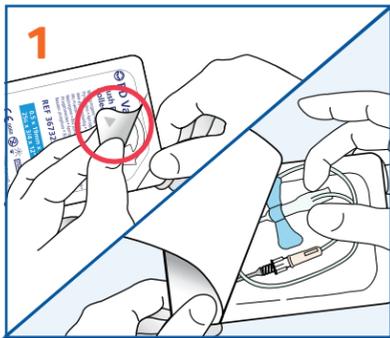

Peel back packaging at arrow so that the back end of the wing set is exposed.


With thumb and middle finger, grasp the rear barrel of the wing set and remove from package. Be careful to avoid activating the button.

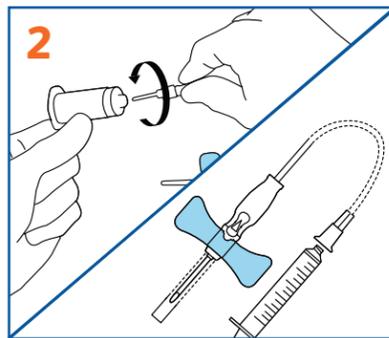


Assemble the wing set to the BD Vacutainer® One-Use holder or BD Syringe.

Apply tourniquet, disinfect the venepuncture site and remove needle sheath.


Never use a blood collection set without a holder or syringe attached.

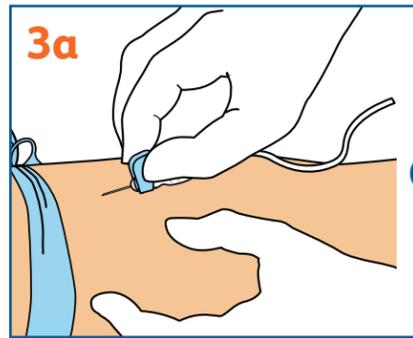

With thumb and index finger, grasp the wings together and access vein using standard needle insertion technique.

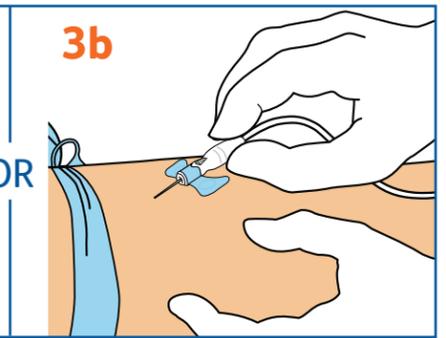


If preferred by your institution, the body of the device can be held, instead of the wings, during insertion. Be careful to avoid activating the button.

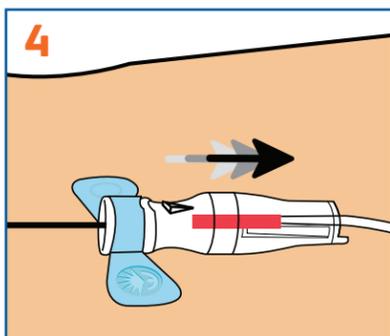

Proper access to the vein will be indicated by the presence of "flash" directly behind and below the button.


Collect specimen(s) in accordance with the tube manufacturer's order of draw and recommended fill volumes and/or the blood culture bottle manufacturer's instructions for use.

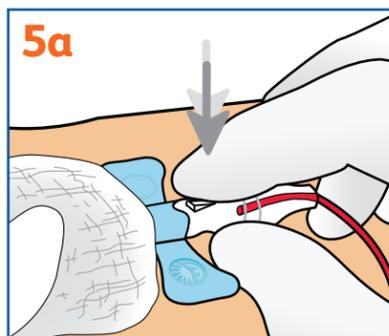


The device is designed to be activated while the needle is still in the patient's vein.

Place your gauze pad on the venepuncture site. Allow gauze pad to cover nose of front barrel. Following the collection procedure, and while the needle is still in the vein, grasp the body with the thumb and middle finger.


Gently activate the button with the tip of the index finger. The needle will automatically retract from the venepuncture site, confirmed by an audible "click".

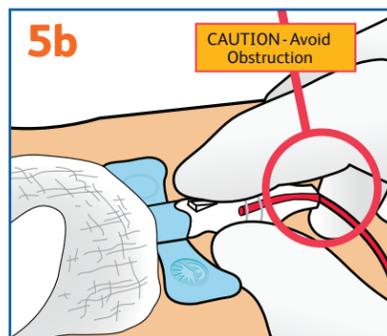


To ensure complete and immediate needle retraction, keep fingers and hands away from the place where the back end of the blood collection set meets the tubing.

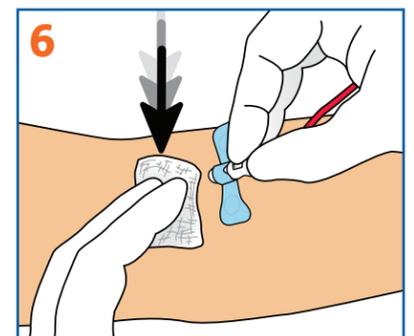

Apply pressure to the venepuncture site in accordance with your facility's protocol.

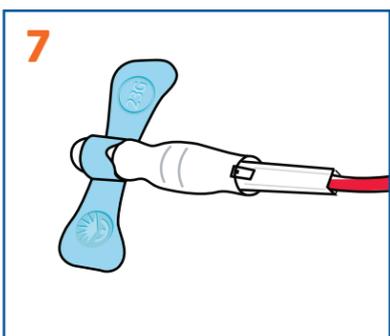

For user safety, confirm that the needle is in the shielded position prior to disposal.

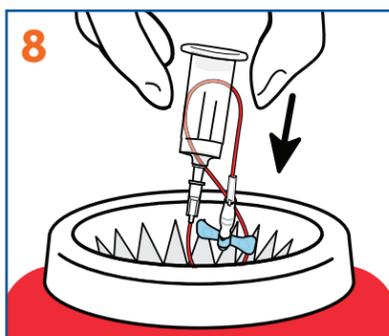

Discard the entire shielded blood collection set and holder into an approved sharps collector.

### CAUTION:

This guideline provides a general overview of use and disposal information. Please follow the policies and procedures of your facility and refer to the Manufacturer's Instructions for Use.

The BD Vacutainer® UltraTouch™ Push Button Blood Collection Set is intended for venepuncture to obtain blood specimens into evacuated tubes and blood culture bottles, as well as for short-term, single infusions up to 2 hours.

## Ordering information

Catalogue No.	Description	Wing colour	Shelf pack/case
367393	21G X.75" needle with 7" tubing with Luer Adapter		50/200
367365	21G X.75" needle with 12" tubing with Luer Adapter		50/200
367392	23G X.75" needle with 7" tubing with Luer Adapter		50/200
367364	23G X.75" needle with 12" tubing with Luer Adapter		50/200
367391	25G X.75" needle with 7" tubing with Luer Adapter		50/200
367363	25G X.75" needle with 12" tubing with Luer Adapter		50/200

Becton Dickinson Pty Ltd, Australia, Toll Free: 1800 656 100  
Becton Dickinson Limited, New Zealand, Toll Free: 0800 572 468

[bd.com/en-anz](http://bd.com/en-anz)

